

ACECUT® Automatic Biopsy System

ACECUT is a fully disposable automatic biopsy system developed for taking a range of tissue specimens including those from the breast, prostate, liver and kidney. Designed to the highest specification ACECUT allows samples to be taken using a simple single-handed operation.

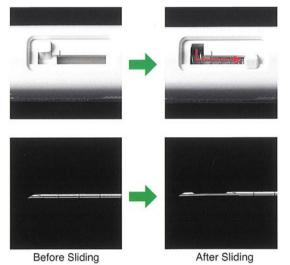

Safe and Easy Specimen Removal

- By sliding the sampling lever backwards
 (as illustrated on the left) the sample can be removed by a simple one-handed operation
- Don't ever lose a sample

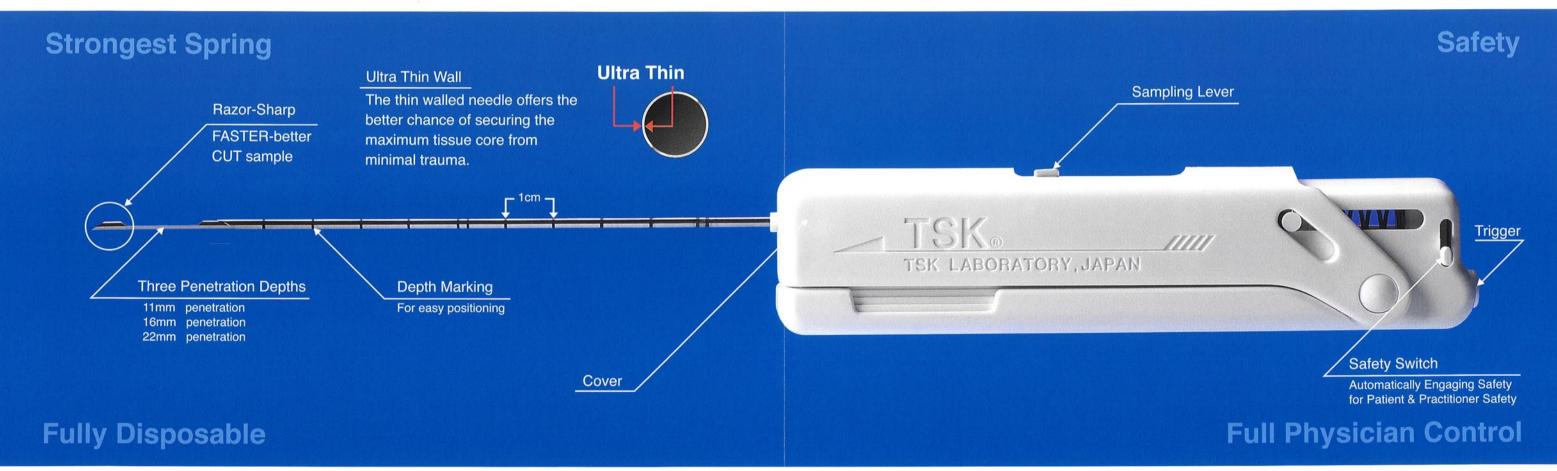

1G UP Co-Axial Guide Needle

For a perfect fit, less traumatic & Better Biopsy





Full Control

- "2 stage" or "Stepped" firing button allows the physicians choice of SEMI-AUTOMATIC or Fully AUTOMATIC firing.
- Semi-Automatic firing allows:
- 1. The Sample Notch to be forwarded and the position of the Notch to be verified under Ultrasound.
- 2.An Ultrasound Image to be taken, saved/printed for post-procedure verification.
- 3.The Sample Notch to be adjusted forward or back (22mm throw) to center in the suspect tissue.
- 4. Allows time for the tissue to settle in the sample notch before cutting for a full sample notch; larger biopsy specimens.

Alternatively, the physician can choose Automatic firing for a shorter procedure.

